



## Sonic Boom Travel & Bed side Alarm Clock with Bed Shaker Model SB200ss



- IMPORTANT** – Please read these instructions carefully before use and retain for future reference. See operating instructions on back.
- Use only the supplied or recommended Sonic Alert adaptor to connect the unit.

- Warning:** To prevent fire or shock hazard do not expose this appliance to rain or moisture. Before connecting to mains make sure that the requirements specified on the apparatus corresponds to the available power supply i.e. AC100 –240V~.
- The ventilation should not be impeded by covering the ventilation openings with items, such as newspaper, table cloths, curtains, etc;
  - No naked flame sources, such as lighted candles, should be placed on the apparatus.
  - It shall not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on the apparatus.
  - Do not place the product in closed bookcases or racks without proper ventilation.

### OPERATION INSTRUCTIONS

**IMPORTANT-** please read these instructions carefully before use and retain for future reference.

	<table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th colspan="2" style="text-align: left; padding: 5px;">FEATURES</th> </tr> </thead> <tbody> <tr> <td style="width: 50%; padding: 5px;">1. Alarm 1 indicator (AL1)</td> <td style="width: 50%; padding: 5px;">9. Snooze Button/ Snooze Duration Adjustment/ Dimmer Control</td> </tr> <tr> <td style="padding: 5px;">2. Alarm Mode Switch</td> <td style="padding: 5px;">10. Tone Control</td> </tr> <tr> <td style="padding: 5px;">3. PM indicator</td> <td style="padding: 5px;">11. Alarm Selector</td> </tr> <tr> <td style="padding: 5px;">4. Alarm Off Button</td> <td style="padding: 5px;">12. Alarm 2 indicator (AL2)</td> </tr> <tr> <td style="padding: 5px;">5. Alarm Set Button / Alarm Duration Adjustment</td> <td style="padding: 5px;">13. Volume Control</td> </tr> <tr> <td style="padding: 5px;">6. Fast Button/ 12 /24 hour Selector</td> <td style="padding: 5px;">14. DC 9V input</td> </tr> <tr> <td style="padding: 5px;">7. Slow Button</td> <td style="padding: 5px;">15 . Vibrator input</td> </tr> <tr> <td style="padding: 5px;">8. Time Set Button</td> <td style="padding: 5px;">16. Battery Compartment</td> </tr> </tbody> </table>	FEATURES		1. Alarm 1 indicator (AL1)	9. Snooze Button/ Snooze Duration Adjustment/ Dimmer Control	2. Alarm Mode Switch	10. Tone Control	3. PM indicator	11. Alarm Selector	4. Alarm Off Button	12. Alarm 2 indicator (AL2)	5. Alarm Set Button / Alarm Duration Adjustment	13. Volume Control	6. Fast Button/ 12 /24 hour Selector	14. DC 9V input	7. Slow Button	15 . Vibrator input	8. Time Set Button	16. Battery Compartment
FEATURES																			
1. Alarm 1 indicator (AL1)	9. Snooze Button/ Snooze Duration Adjustment/ Dimmer Control																		
2. Alarm Mode Switch	10. Tone Control																		
3. PM indicator	11. Alarm Selector																		
4. Alarm Off Button	12. Alarm 2 indicator (AL2)																		
5. Alarm Set Button / Alarm Duration Adjustment	13. Volume Control																		
6. Fast Button/ 12 /24 hour Selector	14. DC 9V input																		
7. Slow Button	15 . Vibrator input																		
8. Time Set Button	16. Battery Compartment																		

## Test Mode:

- 1) Press and hold the Snooze (9) and Time Set button (8) for 3 seconds to enter the Test Mode. The display will show "7ES7" to indicate it is in TEST Mode.
- 2) If within 1 minute no button is pressed, then it will exit the Test Mode and return back to the Time Mode.
- 3) Press any button within 1 minute (except the Snooze & Time button, which has no effect), it will exit the Test Mode immediately.
- 4) While in Test Mode, sliding the Function Switch will demonstrate the different functions of the clock:
  - a) Function Switch at OFF, it will have a continuous "Vee" sound to indicate it is in the Test Mode.
  - b) Function Switch at BUZZ, it will demonstrate the buzzer sound.
  - c) Function Switch at VIB, it will demonstrate the vibrator.
  - d) Function Switch at VIB & BUZZ, it will demonstrate the buzzer sound & the vibrator.
- 5) After finish with the Test Mode, press any button (except the Snooze & Time buttons, which has no effect) it will exit the Test Mode immediately.

## Setting the correct time –

- 1) Press and hold the **"TIME" Button (8)** and at the same time press the **"FAST" Button (6)** or the **"SLOW" button (7)** to advance clock display until you have desired time.
- 2) The **lighted dot (3)** located in upper left corner of the clock display indicates the time is PM. For AM time, this dot will not light up.

## Setting the Alarm Time–

### Setting Alarm 1

- 1) Slide the **"ALARM SELECTOR" (11)** to AL1 position, the alarm indicator **"AL1" (1)** will light up.
- 2) Press and hold the **"ALARM SET" (5)** button and at the same time press the **"FAST" Button (6)** or the **"SLOW" button (7)** to advance clock display until you reach desired alarm setting.
- 3) Make sure alarm time is correctly set.

### Setting Alarm 2

- 4) Once alarm AL1 has been set, slide **"ALARM SELECTOR" (11)** to AL2 position, the alarm indicator **"AL2" (12)** will light up.
- 5) Press and hold the **"ALARM SET" (5)** button and at the same time press the **"FAST" Button (6)** or the **"SLOW" Button (7)** to advance clock display until you reach desired alarm setting.

## Selecting 12 or 24 hours format

- **For Europe, the clock is set to 24 hour format, while for UK the clock is set to 12 hour format.**  
Press and hold the **Fast Button (6)** for 6 seconds to enter the hour selection mode, when the display shows "12 H" or "24 H" release the **Fast Button (6)**. Press and release the **Fast Button (6)** to toggle between 12 or 24 hours. If no selection is made within 4 seconds, then the last displayed hour format will be chosen and the display will return to the normal Time mode.

## Alarm –

### Enable Alarms:

- 1) To enable Alarm 1, slide the **"ALARM SELECTOR" (11)** to AL1 position. The alarm indicator **"AL1" (1)** will light up.
- 2) To enable Alarm 2, slide the **"ALARM SELECTOR" (11)** to AL2 position, the alarm indicator **"AL2" (12)** will light up.
- 3) To enable both alarms, slide the **"ALARM SELECTOR" (11)** to AL1+AL2 position, the alarm indicators **"AL1" (1) and "AL2" (12)** will light up.

## To Select Wake up mode

To select the wake up mode, slide the **"Alarm Mode Switch" (2)** to your desired position:-

- a) "BUZZ" for sound
- b) "VIB" for bed shaker
- c) "VIB/ BUZZ" for both sound and bed shaker

**NOTE:** once the wake up mode is chosen, this will apply to both alarms.

## **Alarm Patterns –**

When the alarm is activated, alarm pattern of the buzzer / flashing display will be out of sync with the vibrator. This means that when the buzzer / flashing display is ON, the vibrator will be OFF and while the buzzer/ flashing display is OFF, the vibrator will be ON.

**ALARM 1** - When Alarm 1 is activate, it will have 3 beeps for buzzer, then the vibrator. This pattern is repeated until the alarm is stopped.

**ALARM 2** - When Alarm 2 is activate, it will have 4 beeps for buzzer, then the vibrator. This pattern is repeated until the alarm is stopped.

## **Adjusting the Alarm Duration Time**

The default alarm duration time setting is 30 minutes; you may adjust the alarm duration between 1 minute and 59 minutes as desired. To adjust the alarm duration:

- Press and hold the **AL. SET button (5)** for six seconds. The clock display will show the default alarm duration as '30'. Release the **AL SET button (5)**.
- Press the **“SLOW” button (7)** to adjust the alarm duration to the desired setting, from 1 minute to 59 minutes.
- When the display is showing the desired alarm duration time, it will return to the normal clock mode after 3 seconds. The new alarm duration time setting becomes the new default and remains in the memory until you change it again.

## **Snooze Operation –**

1. When the alarm turns on in the morning you may press the **SNOOZE Button (9)** to stop the alarm temporarily. The alarm will shut off for 9 minutes or for the predetermined time you select from 1-30 minutes, and then turn on again. The snooze operation may be repeated indefinitely until the alarm is turned off.
2. The SNOOZE function of Alarm 1 and Alarm 2 is independent of each other.

## **Adjusting the Snooze Time**

The default snooze time setting is 9 minutes but you may adjust the snooze time between 1 minute and 30 minutes as desired. To adjust the snooze time:

- Press and hold the **SNOOZE Button (9)** for four seconds. The clock display will show the default snooze time '9'. Release the snooze button
- Press the **“SLOW” Button (7)** to adjust the snooze time to the desired setting, from 1 minute to 30 minutes.
- When the display is showing the desired snooze time, after 3 seconds it will return to the normal clock mode. The new snooze time setting becomes the new default and remains in the memory until you change it again.

## **Shutting Off the Alarm –**

Alarm can be shut off temporary for the day or permanently shut off.

- 1) When alarm goes off, press the **“ALARM OFF” (4)** button to turn off for the day and come back on at the same time the next day, if alarm settings have not changed.
- 2) To shut the alarm off permanently, slide the **“Alarm Mode Switch” (11)** down to the OFF position. The alarm will not come on until the **“Alarm Mode Switch” (11)** is set to **“VIB/BUZZ”, “VIB”, or “BUZZ”** position.

**NOTE:** The Alarm Off function of Alarm 1 and Alarm 2 is independent of each other.

## **Dimmer Operation**

There is a 5-level brightness control for the LED display. Press the SNOOZE button (9) once to change the display brightness. The brightness will cycle from 5-4-3-2-1-2-3-4-5 (from bright to dim then from dim to bright).

**\*NOTE\*:** The dimmer function will not work, when the clock is in snooze mode.

## **Volume Control –**

- 1) To adjust the volume, rotate **VOLUME wheel (13)** located on the lower right corner of the clock.

## Tone Control –

- 1) To adjust the TONE, turn the **TONE wheel (10)** located on the top right corner of the clock.

## Bed Shaker / Vibrator – See Figure B

- 1) Attach the bed shaker jack to the back of clock at “**VIBRATOR INPUT**” (15).
- 2) To turn on vibrator, slide “**Alarm Mode Switch**” (2) to “VIB/BUZZ” or “VIB” position.
- 3) Place vibrator between mattress and box spring.

## Battery Backup –

To use the battery backup, you can install a 9-volt alkaline battery in the back of the clock located under the **Battery Compartment Cover (16)**. The battery needs to be purchased separately. ***Note the LED display will not show during power failure but the clock will still be running and keeping proper time until the power comes back.*** During a power failure, the alkaline battery can only keep the clock IC running for up to 30 days. ***(During a Power failure, the LED display will not show the time.)***

**NOTE:** For the best results it is recommended that the battery be changed every 6 months. If the clock is not used for a while, it is recommended to remove the battery to avoid damage to the clock from battery leakage.

## DC 9V Power Input –



Attach the “**DC POWER INPUT**” (14) to the back of DC9V IN next to “**VIBRATOR INPUT**” (15); it is the smaller input of the two.

**NOTE:** IMPORTANT: do not confuse “**DC POWER INPUT**” (15) with the “**VIBRATOR INPUT**” (15)


## **GENERAL DATA**

UL listed Power Adaptor Input: AC100-240V~, 50/60Hz

Power Consumption: Adaptor Output: DC9V  , 650mA

 <p>The Lightning flash and arrowhead within the triangle is a warning sign alerting you of dangerous voltage inside the product.</p>	<p><b>CAUTION:</b></p> <p>TO REDUCE THE RISK OF ELECTRICAL SHOCK, DO NOT REMOVE THE COVER (OR BACK) FROM THE PRODUCT. NO USER SERVICEABLE PARTS INSIDE. REFER SERVICING TO YOUR SONIC ALERT DEALER.</p>	 <p>The exclamation mark within the triangle is a warning sign alerting you of important instructions accompanying the product.</p>
--	---	--

**WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS APPLIANCE TO RAIN OR MOISTURE.**

- Attention should be drawn to the environmental aspects of battery disposal.
-  This icon indicates separate disposal for electrical and electronic equipment.


Normal function may be disturbed by strong electromagnetic interference. If this is the case, normal function can be resumed by simply resetting the unit following the procedure in the instructions manual. If function does not resume, try moving the unit to another location.

## ***SONIC ALERT'S 1 YEAR LIMITED WARRANTY***

The Sonic Boom Alarm Clock Model SB200ss is warranted against manufacturing defects in materials and workmanship for one (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SB200ss without charge for parts and labor. Simply contact our customer service department to obtain an RA (return authorization number) and instructions on how to process your product for return. Call 1-888-864-2446 or e-mail to [RMA@sonicalert.info](mailto:RMA@sonicalert.info). You'll need a copy of your receipt to process the return authorization.

### SB200ss Cable Support

#### **SBB200ss Cable Support**

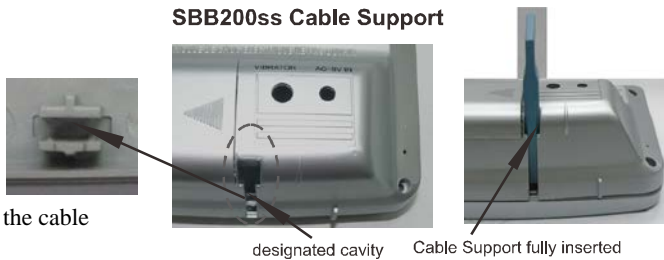

To insert the cable support into the back of the unit, push cable support straight into slot shown. Make sure support is not at an angle; push until it clicks into the designated slot. Please note: once When support is inserted it cannot be removed.

Made in China

Sonic Alert ,Troy, MI 48083

[www.SonicAlert.com](http://www.SonicAlert.com) 1-888-864-2446 V/TTY